


Twist SST Screw-In Suture Anchors and Twist Knotless DEX Screw-In Suture Anchors


Parcus Medical has joined **Anika**





Twist SST Screw-In Suture Anchors and Twist Knotless DEX Screw-In Suture Anchors

* Perform a proper midline incision and dissect the Achilles tendon distally, taking care to leave some medial and lateral distal attachments if feasible. Remove the Haglund deformity with sagittal saw and osteotome. Take care to address any resulting medial and lateral prominences.

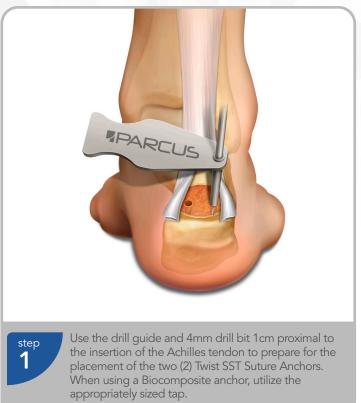


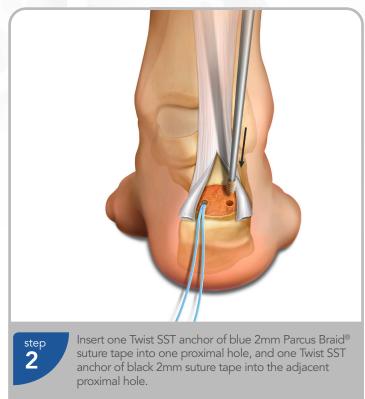


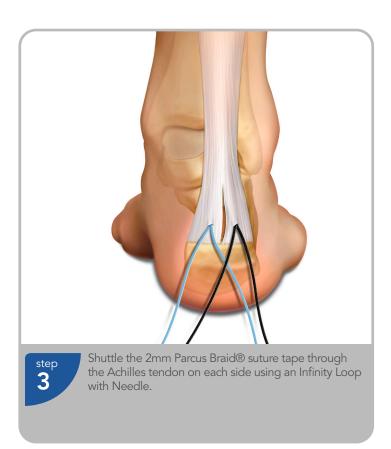


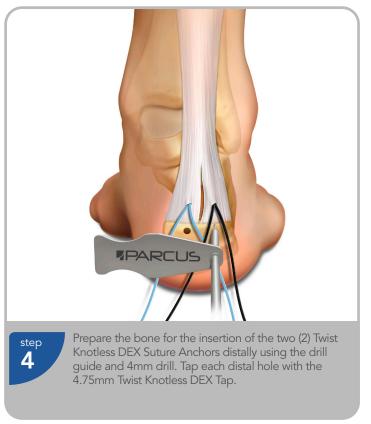


Twist SST Screw-In Suture Anchors and Twist Knotless DEX Screw-In Suture Anchors

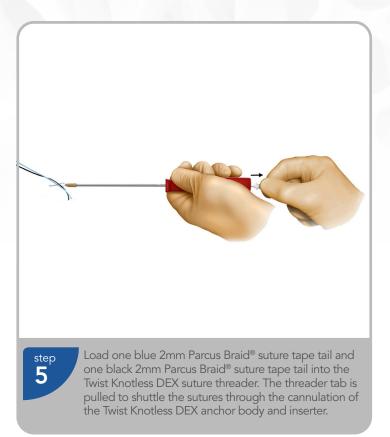


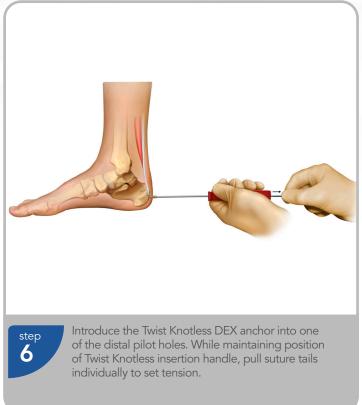

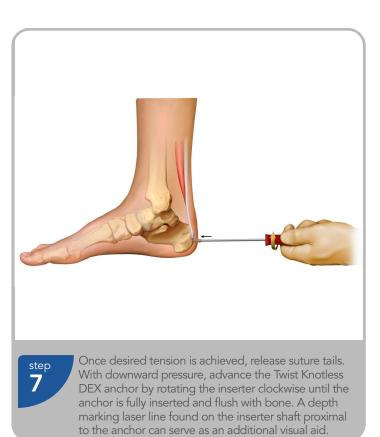

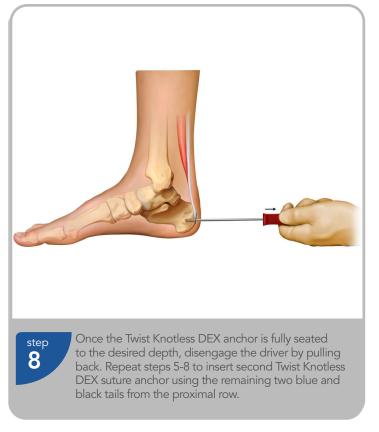

Twist SST Screw-In Suture Anchors and Twist Knotless DEX Screw-In Suture Anchors

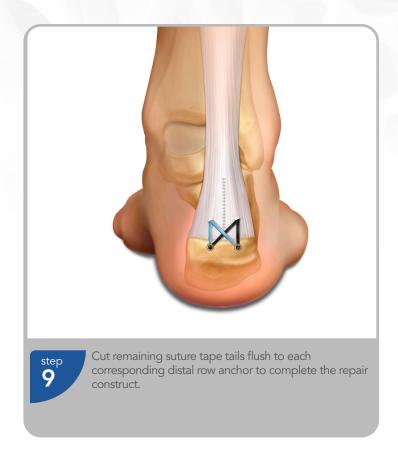

Twist SST PEEK Screw-In Suture Anchor



Twist PEEK Screw-In Suture Anchors are recommended for use in both large and small-joint repairs.

Features & Benefits

Fully-threaded design

- Optimal cortical fixation
- · Reduces the risk of "pull-back"

Made of PEEK-OPTIMA Natural

- Non-absorbable
- Bio-inert*
- Radiolucent
- MR safe

Twist SST PEEK Screw-In Suture Anchors w/One Static 2mm Suture Tape				
Part #	Diameter (mm)	Description	Material	
11349	4.5	Twist SST, screw-in, suture anchor, w/1 static 2mm suture tape (blu)	PEEK	
11350	4.5	Twist SST, screw-in, suture anchor, w/1 static 2mm suture tape (blk)	PEEK	
11351	5.5	Twist SST, screw-in, suture anchor, w/1 static 2mm suture tape (blu)	PEEK	
11352	5.5	Twist SST, screw-in, suture anchor, w/1 static 2mm suture tape (blk)	PEEK	

Parcus Braid Infinity Loop			
Part #	Size	Description	
20056S	#2	Parcus Braid, infinity loop (wht/blu), w/needle	
20174S	1.6mm	Parcus Braid, suture tape infinity loop (wht/blu) w/needle	

Twist SST PEEK Screw-In Suture Anchors Instrumentation			
Part #	Description	Sterility	Single-use/ Reusable
11000U	4.0mm Drill (for use with Twist SST and Knotless DEX Anchors)	Non-Sterile	Reusable
11241	4.0 x 100mm drill guide, 6-point	Non-Sterile	Reusable
10365	3.2mm Awl (for use with 4.5mm Twist SST Anchor)	Non-Sterile	Reusable
10338	4.1mm Awl (for use with 4.5mm, 5.5mm and 6.5mm Twist SST Anchors)	Non-Sterile	Reusable
10609	Punch/tap for 4.5mm Twist SST Anchor	Non-Sterile	Reusable
10277	Punch/tap for 5.5mm or 6.5mm Twist SST Anchors	Non-Sterile	Reusable
10222	Punch/tap for 5.5mm or 6.5mm Twist SST Anchors, hudson	Non-Sterile	Reusable



Twist Biocomposite SST Screw-In Suture Anchor



Twist Biocomposite Screw-In Suture Anchors are recommended for use in both large and small-joint repairs. The proven design of the Twist family has been updated with Parcus Biocomposite material that is comprised of 70% PLGA and 30% B-TCP.

Features & Benefits

Fully-threaded design

- Optimal cortical fixation
- · Reduces the risk of "pull-back"

Made of Parcus Biocomposite material (70% PLGA and 30% B-TCP)


- Resorbable
- Osteoconductive
- Radiolucent
- MR safe

Twist Biocomposite SST Suture Anchors w/One Static 2mm Suture Tape				
Part #	Diameter (mm)	Description	Material	
11355	4.5	Twist Biocomposite SST, suture anchor, w/1 static 2mm Suture Tape, (Blu)	Biocomposite	
11356	4.5	Twist Biocomposite SST, suture anchor, w/1 static 2mm Suture Tape, (Blk)	Biocomposite	
11357	5.5	Twist Biocomposite SST, suture anchor, w/1 static 2mm Suture Tape, (Blu)	Biocomposite	
11358	5.5	Twist Biocomposite SST, suture anchor, w/1 static 2mm Suture Tape, (Blk)	Biocomposite	
11359	6.5	Twist Biocomposite SST, suture anchor, w/1 static 2mm Suture Tape, (Blu)	Biocomposite	
11360	6.5	Twist Biocomposite SST suture anchor w/1 static 2mm Suture Tape (Blk)	Riocomposite	

Twist Biocomposite SST Instrumentation				
Part #	Description	Sterility	Single-use/ Reusable	
11000U	4.0mm Drill (for use with Twist SST and Knotless DEX Anchors)	Non-Sterile	Reusable	
11241	4.0 x 100mm drill guide, 6-point	Non-Sterile	Reusable	
11177	Tap for 4.5mm Twist Biocomposite	Non-Sterile	Reusable	
11178	Tap for 5.5mm Twist Biocomposite	Non-Sterile	Reusable	
11179	Tap for 6.5mm Twist Biocomposite	Non-Sterile	Reusable	
10365	3.2mm Awl (for use with 4.5mm Twist SST Anchor)	Non-Sterile	Reusable	
10338	4.1mm Awl (for use with 4.5mm, 5.5mm and 6.5mm Twist SST Anchors)	Non-Sterile	Reusable	
11433	Soft Bone Tap for 4.5mm Twist Biocomposite	Non-Sterile	Reusable	



Twist Knotless DEX Screw-In Suture Anchors

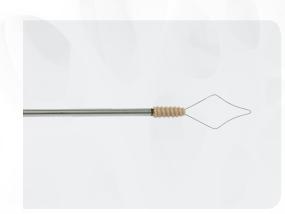

Twist Knotless DEX was designed to be extremely reliable and strong. The fully threaded screw-in knotless anchor is ideally suited for Achilles repair.

Features & Benefits

Knotless

- Provides a step-saving alternative to conventional "knotted" suture anchors
- Eliminates "knot stacks" associated with soft tissue irritation

Fully threaded design

- Provides cortical fixation
- Reduces the risk of "pull-back"

Made of PEEK-OPTIMA Natural

- Non-absorbable
- Bio-inert
- Radiolucent
- MR safe

Twist Knotless DEX			
Part #	Diameter (mm)	Description	Material
11259	4.75	Twist Knotless DEX, screw-in, suture anchor, w/suture passer	PEEK
11260	5.5	Twist Knotless DEX, screw-in, suture anchor, w/suture passer	PEEK

Twist Knotless DEX Instrumentation				
Part #	Description	Sterility	Single-use/ Reusable	
11000U	4.0mm Drill (for use with Twist SST and Knotless DEX Anchors)	Non-Sterile	Reusable	
11241	4.0 x 100mm drill guide, 6-point	Non-Sterile	Reusable	
10338	4.1mm Awl (for use with Twist Knotless DEX Suture Anchors)	Non-Sterile	Reusable	
11139	Punch/tap for 4.75mm Twist Knotless DEX	Non-Sterile	Reusable	





Parcus Medical, LLC. 6423 Parkland Drive, Sarasota, FL 1-941-755-7965 • www.parcusmedical.com

