

Lapiplasty®

Mini-Incision™ System



Experience the Power of Lapiplasty®
Through a **Mini-Incision™** Approach

TREACE®
Medical Concepts, Inc.

The Leader in Hallux Valgus Surgery™

Mini-Incision™

Precision Instrumentation Set

Mini Incision + Triplanar Correction Minus the Compromise

By applying the Mini-Incision™ Positioner over the skin of the 1st metatarsal, the Lapiplasty® Mini-Incision™ System is designed to deliver the patented **Correct Before You Cut** approach for precision 3-plane correction through a 3.5cm dorsal incision.



SpeedSeeker™ Instrument

1405-2377 (Left) 1405-2378 (Right)

Combining the Fulcrum and Joint Seeker functionality for improved stability and a more streamlined approach



Mini-Incision™ Positioner

1405-2383

Enlarged "cup" engagement to distribute the pressure for application over the skin of the 1st metatarsal



Mini-Incision™ Cut Guide

1405-2362 (Gold) 1405-2367 (Platinum)

Miniaturized footprint delivers the benefits of precision bone cuts through a mini incision



Plate Holder

1405-2380

Securely holds Lapiplasty® plates for controlled positioning without excessive tissue retraction




PlantarPower™

Anatomic Tension-Side Plate

Powerful Tension-Side Fixation Designed for a Mini Incision

The PlantarPower™ Plate is uniquely contoured to span plantarly across the tension-side of the 1st TMT joint, while providing easy access to each locking screw without the need for extensive retraction of the mini dorsal incision.

Anatomic
contour
accommodates
tibialis anterior
tendon
insertion

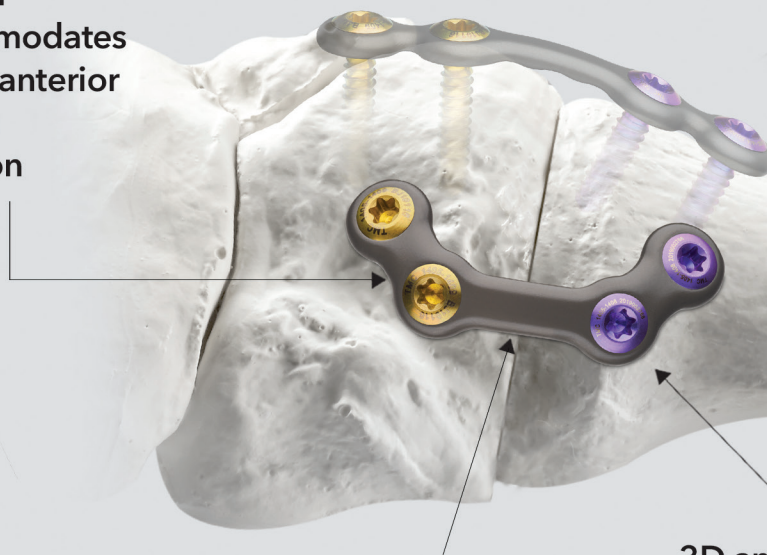

Center span extends plantarly across
tension-side of 1st TMT joint

3D anatomic, U-shaped
curvature for access to
locking screws through
a mini dorsal incision

Ordering Information

SD20 PlantarPower™ Plate*


SK30 Laplasty® Mini-Incision™ System

*Packaged as a single plate (without locking screws)

MIS


Lapiplasty® Mini-Incision™ System

Key surgical steps*

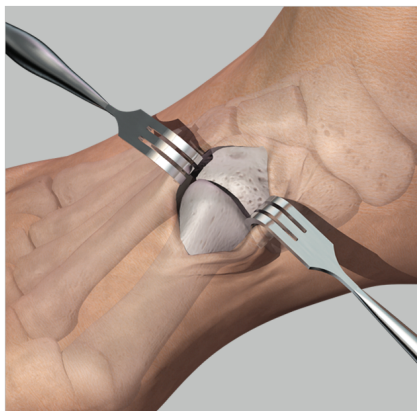


1. Mini Dorsal Incision

Make a direct dorsal, 3.5cm incision and release the 1st TMT joint with a sagittal saw and osteotome.

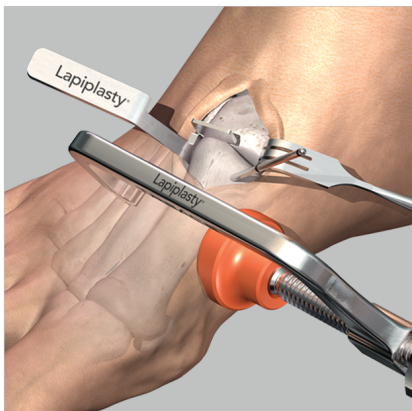


2. Triplanar Correction

Insert **SpeedSeeker™ Instrument** and apply **Mini-Incision™ Positioner** over the skin of 1st metatarsal to perform 3-plane correction.

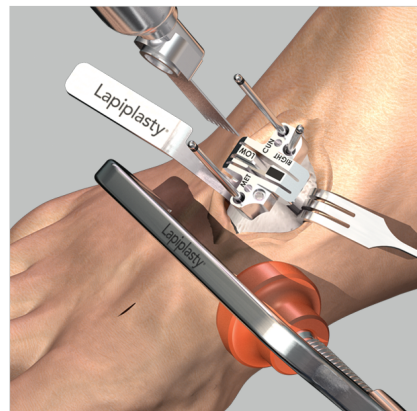


3. Precision Cuts

Secure the **Mini-Incision™ Cut Guide** on the **SpeedSeeker™** and make precise joint cuts with the correction held in place.

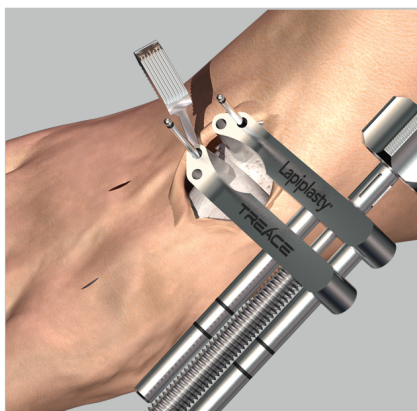

4. Controlled Compression

Apply the **Lapiplasty® Compressor** to bring the precision-cut joint surfaces together for controlled apposition of the arthrodesis site.

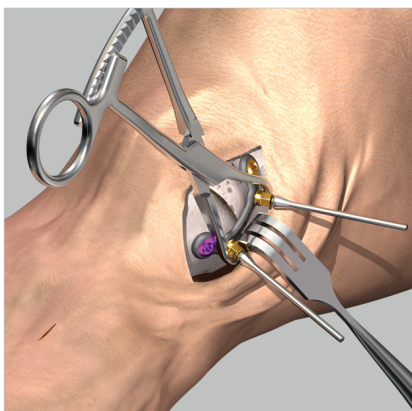

5. Plate Application

Utilize the **Lapiplasty® Plate Holder** to apply the locking plates dorsally and medially, securely positioning the **PlantarPower™ Plate** across the tension-side of the joint.



6. Multiplanar Fixation

Low-profile **Lapiplasty® Biplanar™ Plating** is designed to provide multiplanar stability.

The Leader in Hallux Valgus Surgery™

TREACE
Medical Concepts, Inc.